QUICK GUIDE

GBT MACHINE AIRFLOW® PROPHYLAXIS MASTER

EMS 
MAKE ME SMILE.

**THE
SWISS
ORIGINALS**

-

**BEST
VALUE**

SUMMARY

EN ENGLISH ————— p.4

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TR TÜRKÇE ————— p.178

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WELCOME

Before installation, download & read complete instructions for use (<https://eifu.ems-instruction.com/EMS>).

Use the code FT-300

You can anytime ask for, and receive within 7 days, a free hard copy via our website, by telephone to +800 135 79 135 or in writing to info@e-labeling.eu.

FOLLOW THE INSTRUCTIONS TO INSTALL THE DEVICE

PDF Reader is required. For any further assistance contact EMS aftersales service: +41 (0) 22 99 44 700 - [SSAV@ems-ch.com](mailto:ssav@ems-ch.com).

We thank you for your trust in our GBT Machine AIRFLOW® Prophylaxis Master, which will guide you into the world of GBT - Guided Biofilm Therapy.

The warranty is valid if you complete maintenance 2000 hours after purchase and then 2000 hours after each last maintenance.

For more details, scan the QR code.

**JOIN THE GBT
MOVEMENT NOW!**
gbt-dental.com/gbt-certified

Connections

- + EH-142 Air hose to the cabinet / dental unit.
- + EG-110 Water hose to the cabinet / dental unit.


EM-1009 PIEZON® **Use force**

Power


EH-142 Air **Use force**

EG-110 Water


EM-1007 AIRFLOW® **Use force**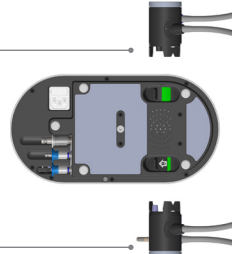

Check cords are locked.

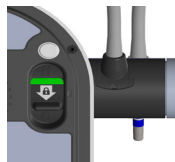←
USE
FORCE

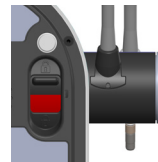Install the batteries in the pedal.

AIRFLOW PLUS: Sub and Supragingival.

AIRFLOW COMFORT: Supragingival on enamel only.

Insert the powder chamber.

The use of the water bottle is optional.

Functions

If the bottle is not placed onto the device then you are connected to the external water supply.

Hereby, Electro Medical Systems S.A., declares that this Medical Device and all of its accessories having radio equipment are compliant with the European Directive 2014/53/EU (RED Radio Equipment Directive). The full text of the EU declaration of conformity is available in the Instruction for Use.

CLEAN

END OF DAY **Clean**

①

Place the NIGHT CLEANER bottle fully filled, switch on the device and lift both cords

②

Hold both cords with CLIP+CLEAN over a sink

③

Press the pedal, and wait for 20 seconds.

④

Disconnect the bottle, unscrew the cap and place it upside down on the device

EACH MORNING **Rinse**

①

Place the water bottle fully filled

②

Hold both cords with CLIP+CLEAN over a sink

③

Press the pedal, and wait for 20 seconds

BETWEEN **Each patient**

①

Clean and disinfect with a wipe (less than 35% alcohol)

EMS recommends the use of either:
 - CleanWipes, IC-100 or PlastiSept Eco Wipes from ALPRO MEDICAL GmbH.
 - Micro-Kleen3™ Surface Disinfectant Wipes from Micro-Scientific
 - CaviWipes™, CaviWipes1™ or CaviWipes™ 2.0 from Metrex

DO NOT use AdvantaClear™ from Hu-Friedy. It damages EMS products.

②

Reprocess your Handpieces and instruments

MAINTENANCE

CHECK IT

Check the blue water filters and change it if needed.

Renew your PIEZON® light guide if needed.

Check instrument wear.

Every 2000 operating hours **Preventive maintenance**

When the wrench symbol on your device is ON, send your device to your authorized EMS repair center.


TROUBLESHOOTING


AIRFLOW® or PERIOFLOW® Handpiece clogged

Use Easy Clean.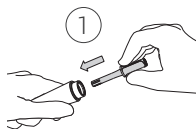

Place the Easy Clean into the handpiece.

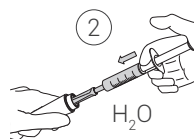Then take a syringe filled with water.

Reprocess your Handpiece.

Service needed
Stop using your device.

Bluetooth
Pair the wireless pedal.

Wi-Fi
Connect your device to Wi-Fi.

Hardware fault
Unplug your device and wait.
If error repeats, call service.

Batteries low
Change pedal batteries.

Upgrade
Launch the software upgrade.

Missing handpiece
Check your handpiece

Missing cord
Check your cords connection.

CONNECTING TO THE WI-FI

ON THE GBT Machine


Press simultaneously the GBT level and high level buttons and hold them for 2 seconds.

A sonar sound will start playing (if not, repeat the step above) and the Wi-Fi symbol will light up and fade several times, connection search is in progress.


ON YOUR Phone/ Computer/ Other

Put your device (phone, computer, etc.) in airplane mode.


 Ensure that only one user is connected to the device's WiFi at a time.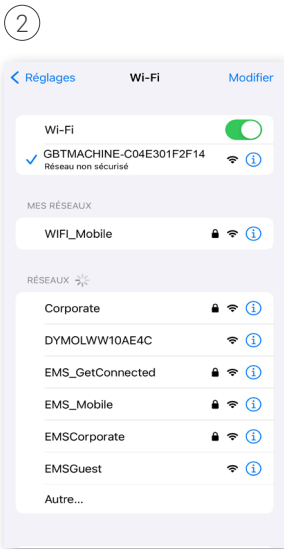

Connect to "GBTMACHINE-XXXX" Wi-Fi network with your phone.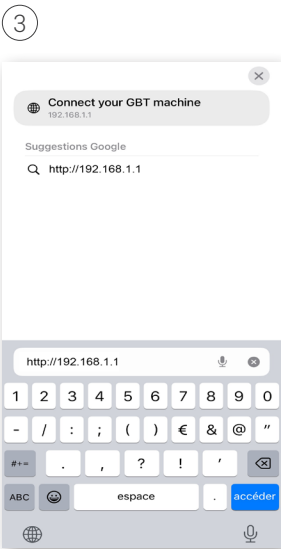

Go on internet browser. Tap <http://192.168.1.1> in the search bar or scan the QR code.

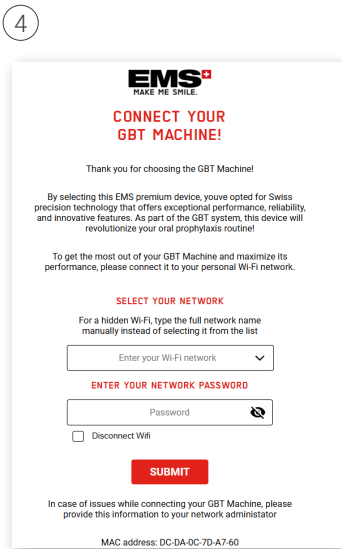Choose your Wi-Fi network or enter the Wi-Fi network name manually (if hidden) and enter your password. If successful, the WIFI logo & the LEDs on the machine will blink twice.

Your GBT Machine is connected to the Wi-Fi !

DISCONNECTING FROM THE WI-FI

ON THE GBT Machine

①

Press simultaneously the GBT level and high level buttons and hold them for 2 seconds.

A sonar sound will start playing (if not, repeat the step above) and the Wi-Fi symbol will light up and fade several times, connection search is in progress.


ON YOUR Phone/ Computer/ Other

Put your device (phone, computer, etc.) in airplane mode.


Ensure that only one user is connected to the device's WiFi at a time.

②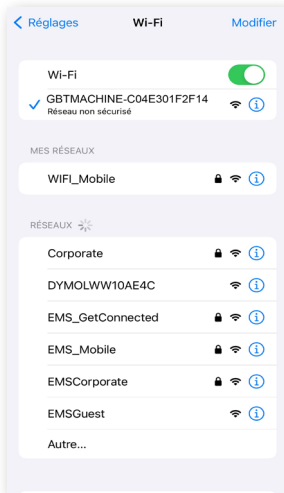

Connect to "GBTMACHINE-XXXX" Wi-Fi network with your phone.


③

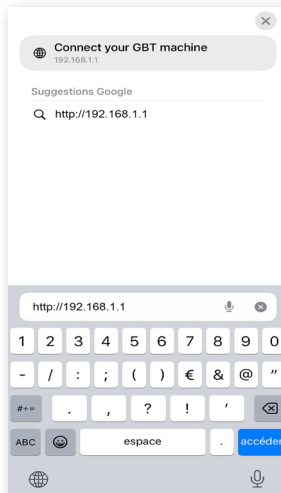Go on internet browser.
Tap <http://192.168.1.1> in the search bar or scan the QR code :

④

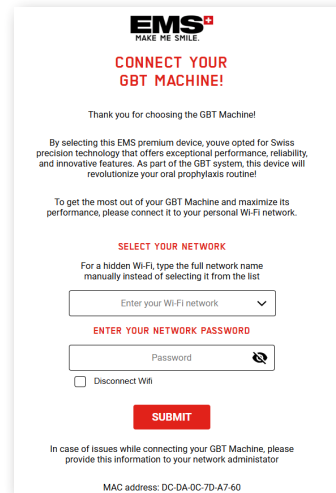Check the Disconnect Wi-Fi box, then click on the Submit button.

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(SHANGHAI) CO., LTD.
EMS ELECTRO MEDICAL SYSTEMS KOREA LTD.